


UC-321PL

Precision Health Scale

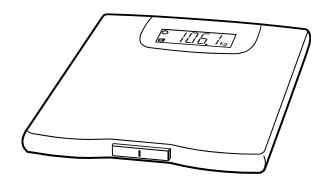

Instruction Manual

Before Use

	Precautions 1		
	Scale Components2		
	Display Symbols 3		
	Installing (or Replacing) Batteries 3		
	Operations		
	Basic Measurement 4		
	Measuring Your Weight on a Carpet 6		
References			
	Specifications7		
	Maintaining the Scale7		
	Before Requesting Service 8		

Precautions

- Please read this booklet before using your scale.
- Place your scale on a solid level surface, such as tile, wood, or laminate flooring.
- Do not place your scale on a soft, wet or slippery floor.

 If the scale must be used on a carpet, be sure to attach the plastic add-on feet as shown on Page 6.
- Your scale contains delicate, high-precision parts. Avoid extremes in temperature, humidity, direct sunlight, shock, moisture and dust.
- Do not step on your scale with your feet wet. Be sure that your feet are dry to prevent slipping or serious injury.
- Step on the center portion of your scale gently.
- Do not step on the display window.
- Holding the measurement switch for extended periods will shorten battery life.

Indications for Use

■ This scale is designed for home and clinical use.










Scale Components

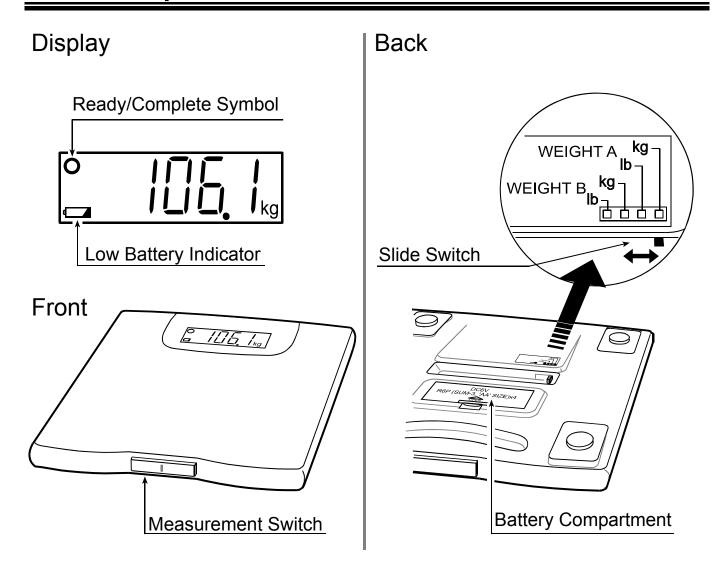

- The following measurements can be selected by the slide switch and are classified into two categories WEIGHT A and WEIGHT B. Within WEIGHT A and WEIGHT B, you have a choice to select and have your weight displayed in kg or lb. The difference between WEIGHT A and WEIGHT B is the speed of communication from the data port to another device (e.g. modem, printer, telephone). Please follow instructions from your telemonitoring company to select the proper position on the slide switch.
- The scale has an "Automatic- power-off" function to turn off power after about 45 seconds of non-operation.

Display Symbols

0	Ready/Complete symbol. This symbol is displayed when the measurement display is stable.
	Low battery indicator.
	If this symbol is displayed, replace all four batteries with new
	ones.


Installing (or Replacing) Batteries

Caution

- Insert each battery with its positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
- Improper installation of the batteries may cause battery leakage.
- Do not mix new and used batteries. It may cause damage to the scale or shorten battery life.
- The battery life is dependant upon the environmental temperature, battery type and usage.
- Remove the batteries from the battery compartment, when the scale is not to be used for a long time. They may leak and damage the scale.

Steps for Installing (or Replacing) Batteries

- Remove the battery compartment cover on the underside of the scale by gently pushing the hook in the direction indicated by the arrow and lifting the cover backward.
- 2. Install four new batteries with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
- 3. Replace the cover by sliding it into the compartment and gently pressing into place.

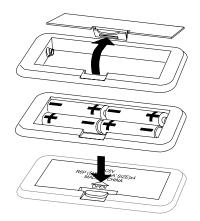

Basic Measurement

Measuring Your Weight

1. Set the slide switch to the correct mode. Example: Set the slide switch to [WEIGHT B].

Set the slide switch to the correct mode



2. Press the measurement switch gently.

All display segments are displayed for several

seconds.

3. Wait until the O symbol is displayed.

Wait until "0.0 kg /0.0 lb" and the Ready/Complete symbol are displayed.

4. Step on the scale gently and stand still during the measurement.


You will see the display changed to dashes. Please remain still as much as you can.

5. Your weight is displayed after the O symbol is displayed.

Note: If "0.0 kg /0.0 lb" is displayed for about 45 seconds, the scale will automatically turn off.

Ready/Complete symbol O Step on the scale Step off the scale

6. Step off the scale.

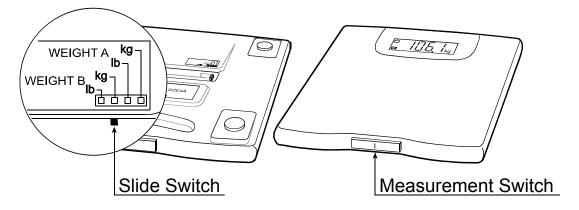

Basic Measurement

Measuring an Infant or Small Animal

- 1. Set the slide switch to the correct mode.

 Example: Set the slide switch to [WEIGHT B].
- 2. Press the measurement switch gently and step on the scale.

All display segments are displayed for several seconds.

Step on the scale without any baby or animal in your arms before "0.0 kg /0.0 lb" is displayed.

Press the measurement switch Step on the scale

Ready/Complete symbol

Have the infant or the animal placed in your arms

Set the slide switch to the correct mode

3. Have the infant or the animal placed in your arms

Remain on the scale and have the infant or the animal placed securely in your arms within ten seconds after the scale displays "0.0 kg /0.0 lb" and the Ready/Complete symbol .

Remain still during the measurement until the Ready/Complete symbol . is displayed. You will see the display changed to dashes. Please

remain still as much as you can.

 The weight of the infant or the animal can be read after the O symbol is displayed.

After about 45 seconds, the scale will automatically turn off.

Step off the scale

5. Step off the scale.

Measuring Your Weight on a Carpet


Place the scale on a solid level surface, such as tile, wood, or laminate flooring. If the scale must be used on a carpet, be sure to attach the plastic add-on feet.

Steps for Attaching the Add-on Feet

- 1. Each add-on foot has two pins. Match those pins to the holes on the scale leg and insert firmly.
- 2. Attach add-on feet on all four legs.

Notes

- Please be careful not to fall when using the add-on feet.
- We recommend using this scale on a solid and level surface.

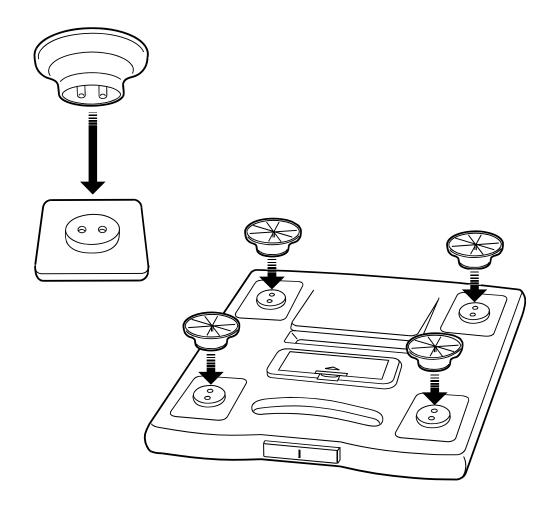

Specifications

Digital display Liquid crystal display, 25 mm character height

Maximum capacity 200 kg / 450 lb

Minimum display 0.1 kg / 0.2 lb

Power source Four type AA batteries

(R6P, 1.5 volt x 4)

Battery life Approximately 1000 measurements

Accessories Add-on feet (4), Instruction manual

Specifications are subject to change without prior notice or any obligation on the part of the manufacturer.

Compliance with European Directives

The scale complies with the statutory EMC (Electromagnetic Compatibility) directive 89/336/EEC.

Notes: Indoor use only.

The displayed value may be adversely affected under extreme electromagnetic influences.

Maintaining the Scale

- Clean the scale with a soft dry cloth or a cloth moistened with water and a mild detergent. Never use alcohol, benzine, thinner or other harsh chemicals to clean the scale.
- Remove the batteries from the battery compartment, when the scale is not going to be used for a long time.

Before Requesting Service

Problems	Corrective Action	
Nothing is displayed by pressing the	Check the battery installation.	
measurement switch.	Replace all four batteries with new	
	ones.	
"88888" remains displayed.	Remove and reinstall the batteries.	
"H" or "L" is displayed.	Remove and remstan the batteries.	
The difference between an estimated	Place the scale on a solid level	
weight and a measured weight is	surface.	
excessive.	Remove any foreign objects that might	
	be touching the bottom of the scale.	
	When the scale is placed on a carpet,	
	be sure that the add-on feet are	
	attached or try moving the scale to a	
	level surface such as tile or wood.	
"E - □" is displayed.	Replace all four batteries with new	
	ones.	

If you can not locate and fix the problem, contact your supplier, or the A&D service group. The A&D service group will support authorized suppliers with technical information, spare parts and units.

A&D Company, Limited

3-23-14 Higashi-Ikebukuro, Toshima-ku, Tokyo 170-0013 JAPAN

Telephone: [81] (3) 5391-6132 Fax: [81] (3) 5391-6148

A&D INSTRUMENTS LTD.

<Authorized Representative Established in the European Community>
Unit 24/26 Blacklands Way, Abingdon Business Park, Abingdon, Oxfordshire OX14 1DY United Kingdom

Telephone: [44] (1235) 550420 Fax: [44] (1235) 550485

A&D MERCURY PTY. LTD.

32 Dew Street, Thebarton, South Australia 5031 AUSTRALIA Telephone: [61] (8) 8301-8100 Fax: [61] (8) 8352-7409